


ASSIST TOILET SEAT

SEAT CODE:


RB6015W - White

RB6015B - Blue

RB6015G - Gray

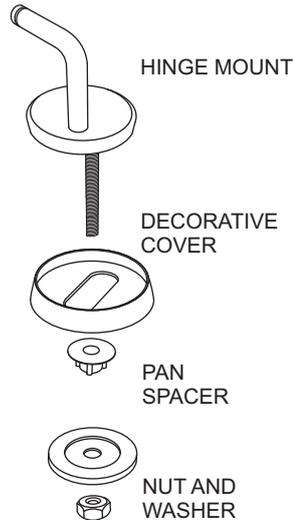

Hinge structure:
(code: XR909)

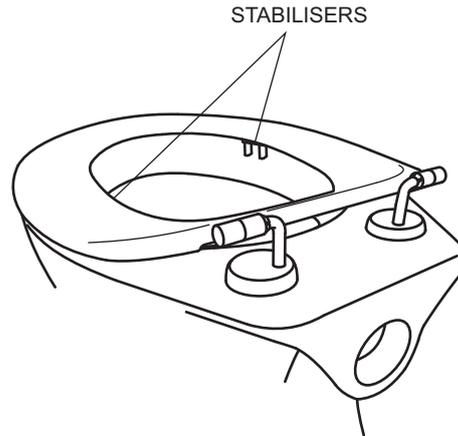

NOTE:

This seat is available in Dark Blue for vision impaired people, in White for assisted living/geriatric applications and also in Gray.

INSTALLATION

1. Fit hinge mount assembly as shown into the seat fittings (Note: make sure to fit the hinges the right way round - the elbow posts should be to the back).
2. Insert Pan Spacer into pan and assemble the seat bolt through Decorative cover and holes in the Pan.
3. Adjust seat so that it fits into position on the bowl and tighten the bolts with nuts, using the spanner tool provided
4. Fit the lateral stabilisers so that they are located correctly just on the inside of the bowl rim tighten and fit the stabilisers' cover pads.

CARE

1. Clean only with a soft cloth using mild soapy warm water
2. Do not use solvent based chemicals
3. Do not use harsh or abrasive cleaners
4. Do not expose to direct sunlight.