



TOILET SEAT INSTALLATION INSTRUCTIONS

These instructions are for 'Life Assist' Toilet seat

1. BEFORE INSTALLATION


- Before installation check that the seat and the hinge bolts are a suitable fit on the toilet bowl.
- Please read these instructions carefully to ensure trouble-free fitting of the seat.
- Please retain the instructions for future reference

2. MAINTENANCE

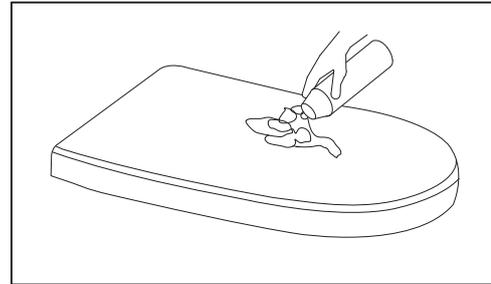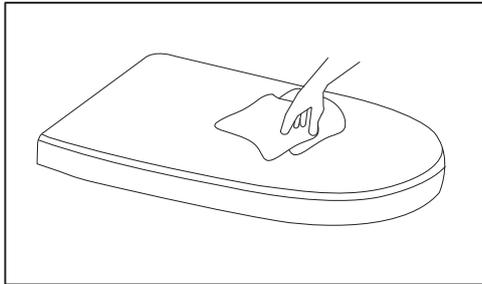


- Regularly wipe the seat using a damp cloth and mild detergent
- For convenience the seat may be removed by depressing the two hinge buttons and lifting the seat up (see Diagram 6 on the next page).
- With the seat removed, the area around the hinge mounts can be easily cleaned.
- Avoid spilling cosmetics and solvents as this can permanently stain the surface finish
- Periodically check the tightness of the hinge fixing screws.


3. CAUTIONS

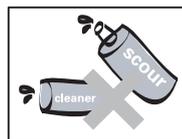

Do not clean the seat or cover with strong chemical cleaners, scouring abrasives or stiff brushes as these may ruin the polished surface.

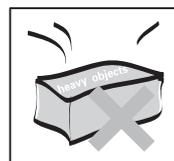

Do not place heavy objects on the cover and avoid dropping items onto the seat.

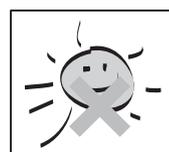


Where possible avoid installing in locations subject to strong sunlight or high temperature.


4. INSTALLATION STEPS

Seat Assembly

1. Insert the top-fix fittings into the seat bolt holes of the pan. Push down firmly.
2. Attach the hinge bases and lightly screw down the fixing bolts.
3. Fit the hinge mechanisms to the seat - Note R and L and ensure that the holes for the fixing screws face to the front of the Pan.
4. Slide the seat and hinge assembly back and check the fit of the seat to the pan.
5. adjust as necessary..
6. Remove seat and tighten fixing bolts firmly. Refit the seat.
7. Insert and tighten the fixing screws.
8. Fit the lateral stabilisers and adjust so that they are located correctly just on the inside of the bowl rim. Fit the stabiliser cover pads.

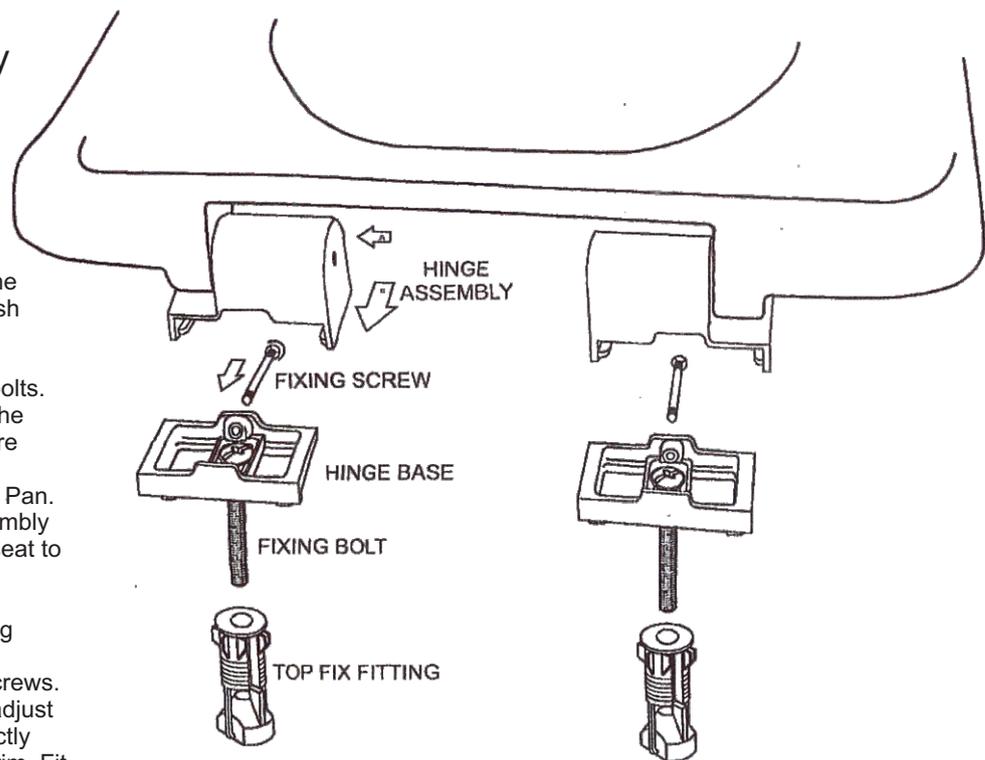

Stabilizers Assembly

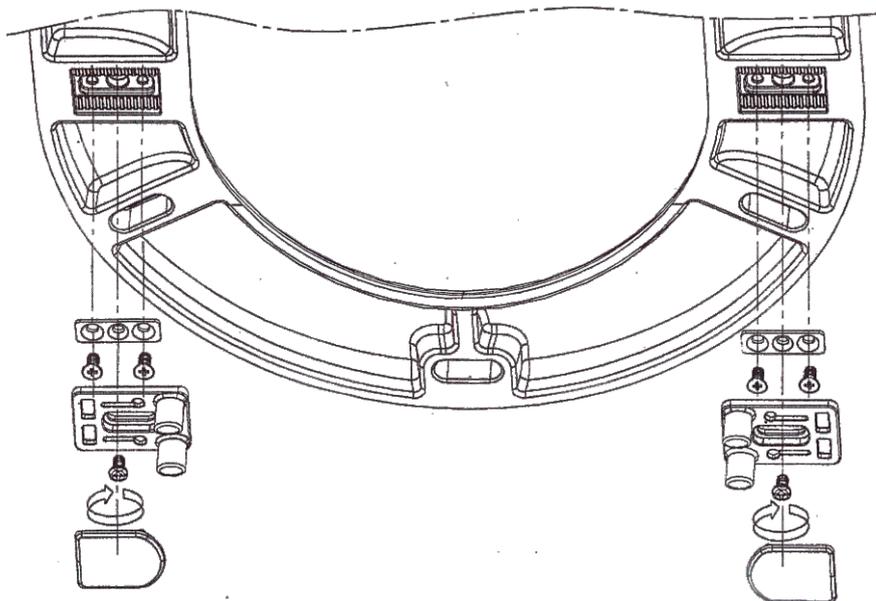

NOTE: This seat is available in Dark Blue for vision impaired individuals.

The seat is also available in White for assisted living/geriatric applications

RB6044B - Blue
RB6044G - Grey
RB6044W - White

WARRANTY
Visit www.johnsonsuise.com.au
for terms and conditions

MANUFACTURED IN CHINA FOR
BATHROOM PRODUCTS AUSTRALIA P/L

ABN 87 079 297 617
2/97 BANKSIA ST. BOTANY, NSW 2019
TEL: 1300 717 717 FAX: 1300 746 480
www.johnsonsuise.com.au